
# Kit Cãnula Simple Block

Reg. ANVISA: 80195520029.

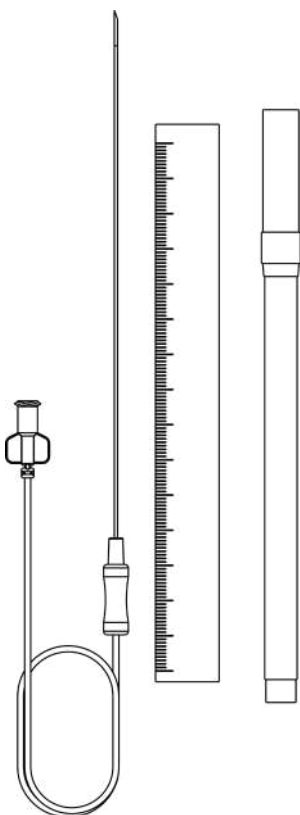## **BRAMSYS**

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## SIMPLE BLOCK CANNULA KIT

### PURPOSE

The cannula is introduced into the patient and its location is determined through the use of fluoroscopy, ultrasound or x-ray in order to confirm the location of the plexus, or peripheral nerves, enabling the block to be performed through the injection of anesthetic fluids/medications.

### INDICATIONS OF USE

The Simple Block Cannula Kit is indicated for carrying out procedures to block the plexus and peripheral nerves using anesthetic solutions/medicines. The Simple Block Cannula Kit should only be used in hospitals, clinics, outpatient clinics and offices, under aseptic conditions and handled by doctors duly qualified and trained to carry out the procedure.

### COMPOSITION

Primary packaging: surgical grade paper envelope containing 1 or 2 cannulas surrounded by a protective tube and 1 surgical planning scale, 1 surgical marker.

Secondary packaging: surgical grade paper envelope or individual packaging box.

Tertiary packaging: transport box.

Code	Ø	Length	Qty Cannulas
SBK1805	18G	50mm	1
SBK1805-2	18G	50mm	2
SBK1810	18G	100mm	1
SBK1810-2	18G	100mm	2
SBK1815	18G	150mm	1
SBK1815-2	18G	150mm	2
SBK1820	18G	200mm	1
SBK1820-2	18G	200mm	2
SBK2105	21G	50mm	1
SBK2105-2	21G	50mm	2
SBK2110	21G	100mm	1
SBK2110-2	21G	100mm	2
SBK2115	21G	150mm	1
SBK2115-2	21G	150mm	2
SBK2120	21G	200mm	1
SBK2120-2	21G	200mm	2
SBK2205	22G	50mm	1
SBK2205-2	22G	50mm	2
SBK2210	22G	100mm	1
SBK2210-2	22G	100mm	2
SBK2215	22G	150mm	1
SBK2215-2	22G	150mm	2
SBK2220	22G	200mm	1
SBK2220-2	22G	200mm	2

### USE INSTRUCTIONS

- Make sure that the product packaging is not damaged.
- Remove the Cannula, surgical planning scale and surgical marker from their sterile packaging in a clean environment using aseptic techniques.
- Make sure that the cannula, surgical planning scale and surgical marker are intact.
- Position the patient according to the surgical procedure to be performed.
- Perform a fluoroscopy or x-ray image.
- Mark the incision site with the surgical planning scale and surgical marker.
- Aspirate the anesthetic solution/medicine to be injected into a syringe.
- Insert the Simple Block Cannula into the patient according to the marking made using the surgical planning scale and surgical marker.
- Fit the syringe containing the anesthetic solution/medication into

the injector tube with luer lock connection to perform the nerve block.

- After completion of the procedure, the material must be destroyed and discarded in an appropriate location, following the regulations applicable to the disposal of sharp and contaminated material, adopted by the hospital

NOTE: The syringe mentioned above is not part of this notification process.

### **STORAGE CONDITIONS**

The Simple Block Cannula Kit must always be stored protected against all types of moisture, dirt and contact with flammable, corrosive, toxic or explosive substances. The packaging must not have holes or tears, and must be kept away from sharp or sharp instruments.

Do not place it under weight or volumes, as they could damage its technical characteristics and compromise its integrity.

Do not store the product where it may be exposed to direct sunlight, high temperatures, humidity or dust to avoid compromising its integrity/quality.

Do not place them next to or next to different categories, as they could cause damage to their technical characteristics and compromise their integrity/quality.

Storage temperature: 5°C to 60°C.

Storage humidity: 10% to 80%.

### **PRECAUTIONS, RESTRICTIONS AND WARNINGS**

- The user, before using the product, must carefully read the instructions for use, which will assist in correct and safe use, as well as protect against possible risks.

- Do not use the product after its expiration date, always check that the sterilization indication is correct.

- The product should only be used by qualified medical professionals, who have the knowledge and experience necessary for the proper use of the product, knowledge of handling sterile products and, above all, the importance of maintaining a completely sterile environment in operating rooms, where care and precaution with every detail are essential so that there is no risk of contamination.

- Check the integrity of the packaging before using the product, in case of violation, DO NOT USE, as the guarantee of sterility will be compromised.

- Totally safe product if used by a qualified professional and respecting the usage specifications described in this document.

- In case of irregularities or problems with the product, contact the manufacturer to take the necessary measures.

- The product must not be reused and/or re-sterilized, being for ONLY use, and must be destroyed and discarded in HOSPITAL WASTE.

- Do not store or store this product close to products from different categories.

- This product is for single use, sterilized by Ethylene Oxide (ETO).

- Reprocessing this product is prohibited.

- When transporting it, do so with caution and in accordance with the standards for transporting sterilized products, as inadequate transport may compromise the continued integrity and quality of the product offered by the manufacturer

- The product must not be reused and/or re-sterilized, being for SINGLE use, and must be destroyed and discarded in HOSPITAL WASTE.

Totally safe product if used by a qualified professional and respecting the usage specifications described in this document.

**MS Registration:** 80195520029

**Technical Responsible:** Marcos Rojas

**CREA:** 5061214879

### **WARRANTY CERTIFICATE**

**BRAMSYS INDÚSTRIA E COMÉRCIO LTDA**, properly authorized by the National Agency of Sanitary Surveillance, guarantees for **24 MONTHS**, starting from the sterilization date, the quality and the sterile condition of this product, except in the cases of bad usage or noncompliance of the orientations written in this document.

**SERVIÇO DE ATENDIMENTO AO CLIENTE (SAC)**  
**SERVICIO DE ATENDIMIENTO AL CLIENTE (SAC)**  
**CUSTOMER SERVICE (CS)**  
Phone: +55 11 5013 0010  
E-mail: [sac@bramsys.com.br](mailto:sac@bramsys.com.br)  
Site: [www.bramsys.com.br](http://www.bramsys.com.br)