
Misturador de Cimento Ósseo

Reg. ANVISA: 80195520040.

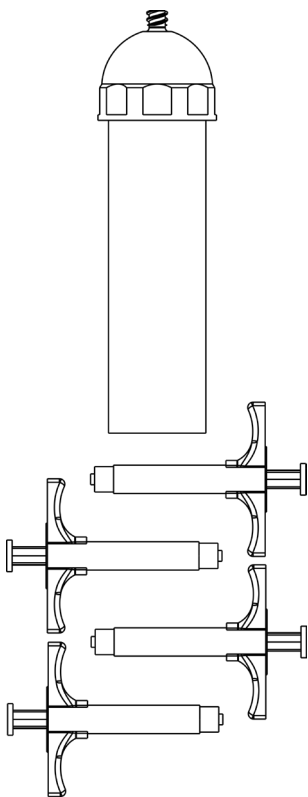BRAMSYS

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BONE CEMENT MIXER

PURPOSE

This product is designed to assist in the preparation of bone cement during surgical procedures.

OPERATING PRINCIPLE

Promote the mixing of the bone cement components by agitating the container with its metal ball and transferring the contents to the winged syringes through its luer lock connection.

COMPOSITION

(4) Winged syringe; (1) Bone cement mixer.

COMMERCIAL PRESENTATIO

Primary Packaging: Surgical pouch.

Secondary Packaging: Individual box.

Code	Winged syringe		Bone cement mixer	
	Capacity	Qty.	Capacity	Qty.
MCO40	2.5ml	4	40g	1

USE INSTRUCTIONS

- Ensure that the packaging is not damaged.
- Remove all items from the sterile packaging in a clean environment using aseptic techniques.
- Ensure all items are intact.
- Open the mixer.
- Insert the bone cement components into the mixer.
- Close the mixer and ensure that the luer lock cap is securely closed.
- Shake the mixer until the ideal viscosity is reached according to the physician's preference.
- Remove the luer lock cap.
- Connect the winged syringe to the mixer's luer lock.
- Turn the bone cement mixer so that the luer lock is facing downward.
- Collect the bone cement with the winged syringe.
- Dispose of all used materials properly after the procedure.

STORAGE AND TRANSPORT CONDITIONS

To maintain the integrity of the product, it must:

Be stored and transported always protected from sunlight, high temperatures, excessive humidity, dirt, and contact with flammable, corrosive, toxic, or explosive substances.

The packaging must not be punctured or torn, and it should be kept away from sharp or pointed instruments.

Do not place under weight or heavy volumes, as this may damage the product's technical characteristics.

Do not place it near or together with different categories, as they may damage its technical characteristics.

It must be stored and transported following good practices for the transport of sterilized products.

This product is sterilized by ethylene oxide (ETO) and the packaging must not be damaged before use.

The product must be transported in its original packaging.

Storage temperature: Between 15°C and 30°C.

PRECAUTIONS, RESTRICTIONS AND WARNINGS

- Avoid using the product if the packaging is damaged.
- Do not use the product if the expiration date has passed.
- The product is intended for single use and must be discarded after use.
- Reuse or resterilization may compromise the structural integrity of the product and/or cause damage that could result in injury or illness to the patient.
- It is important to carefully read the instructions for use and these precautions before using the device.

MS Registration: 80195520040

Technical Responsible: Marcos Rojas

CREA: 5061214879

WARRANTY CERTIFICATE

BRAMSYS INDÚSTRIA E COMÉRCIO LTDA, properly authorized by the National Agency of Sanitary Surveillance, guarantees for **24 MONTHS**, starting from the sterilization date, the quality and the sterile condition of this product, except in the cases of bad usage or noncompliance of the orientations written in this document.

CUSTOMER SERVICE (CS)

Phone: +55 11 5013 0010

E-mail: sac@bramsys.com.br

Site: www.bramsys.com.br