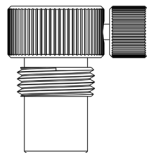
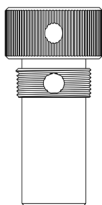
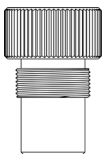
Adaptadores Estereotáxicos

Reg. ANVISA: 80195520007.

BRAMSYS

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STEREOTACTIC ADAPTERS

PURPOSE

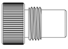
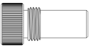
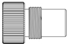
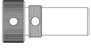
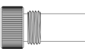
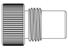
Bramsys Adapters are assembled to serve as a guide or adjustment for electrodes and cannulas in surgical equipment.

INDICATIONS FOR USE

- Bramsys Adapters are indicated for stereotactic surgical procedures where it is necessary to guide and adjust the cannula or electrode in a stereotactic surgical device.
- The instructions for use that accompany the product must be read before opening the sterile packaging to ensure its correct and safe use.

COMPOSITION

3 Adapters. All parts and components of the product are marketed for exclusive use with it.

Model	Compatibility	Image
BUSHING ADAPTER 1.5x23 M12	BMS-D072; BMS-D027; BMS-D029; BMS-D014; BMS-D033; BMS-D034; BMS-D036 (80195520002)	
BUSHING ADAPTER 1.5x31 M12	BMS-D072; BMS-D027; BMS-D029; BMS-D014; BMS-D033; BMS-D034; BMS-D036 (80195520002)	
BUSHING ADAPTER 2.2x23 M12	BMS-D145; BMS-D146; BMS-D150; BMS-D151; BMS-D490 (80195520014) BMS-D149; BMS-D157 (80195520017)	
BUSHING ADAPTER 2.2x31 M12	BMS-D145; BMS-D146; BMS-D150; BMS-D151; BMS-D490 (80195520014) BMS-D149; BMS-D157 (80195520017)	
BUSHING ADAPTER 2.2x23 M12 PAIR	BMS-D152; BMS-D153 (80195520018)	
BUSHING ADAPTER 2.2x31x7,15 M12	BMS-D152; BMS-D153 (80195520018)	
BUSHING ADAPTER 2.6x23 M12	-	
BUSHING ADAPTER 2.6x31 M12	-	
BUSHING ADAPTER 3.2x31 M12	-	
BUSHING ADAPTER 3.2x23 M12	-	

INSTRUCTIONS FOR USE

- Check that the packaging is not damaged or wet.
- Verify that the sterilization indicator is correct.
- Check the sterilization expiration date.
- Open the packaging box and remove the sealed blister.
- Following sterile product handling procedures, open the blister and remove the product.
- Locate the mounting site for the adapters on the equipment.
- Install the adapters in the designated locations, ensuring the correct order.
- Perform the surgical procedure.
- After the procedure, remove the adapters from the equipment. It is recommended to render them unusable and dispose of them in MEDICAL WASTE.

PRECAUTIONS, RESTRICTIONS, AND WARNINGS:

- Before using the product, the user must carefully read the instructions for use, as they will assist in its correct and safe application while also protecting against potential risks.
- Do not use the product beyond its expiration date; always check that the sterilization indicator is correct.
- The product should only be used by qualified medical professionals who have the necessary knowledge and experience for proper use, including sterile product handling and, most importantly, the importance of maintaining a completely sterile environment in operating rooms. Attention to detail is crucial to prevent contamination risks.
- The product must not be reused or resterilized, as it is SINGLE-USE and must be destroyed and disposed of in MEDICAL WASTE.
- The adapters must always be transported and stored protected from any type of moisture, dirt, and contact with flammable, corrosive, toxic, or explosive substances.
- The packaging must not be punctured or torn; keep it away from sharp or pointed instruments.
- Do not store or place this product near products of different categories.
- Check the integrity of the packaging before using the product. If the packaging is compromised, DO NOT USE IT, as its sterility cannot be guaranteed.
- This product is sterilized using Ethylene Oxide (ETO), Hydrogen Peroxide, or Gamma Radiation.
- The resterilization and reuse of this product are not covered by the manufacturer's warranty, thereby exempting the manufacturer from any liability.

This product is completely safe when used by a qualified professional and in accordance with the usage specifications described in this document.

Lot Number, Sterilization Date, and Expiration Date: See product packaging

MS Registration: 80195520007

Technical Responsible: Marcos Rojas

CREA: 5061214879

WARRANTY CERTIFICATE

BRAMSYS INDÚSTRIA E COMÉRCIO LTDA, properly authorized by the National Agency of Sanitary Surveillance, guarantees for **24 MONTHS**, starting from the sterilization date, the quality and the sterile condition of this product, except in the cases of bad usage or noncompliance of the orientations written in this document.

CUSTOMER SERVICE (CS)

Phone: +55 11 5013 0010

E-mail: sac@bramsys.com.br

Site: www.bramsys.com.br