# Cânula para Cirurgia Guiada

Reg. ANVISA: 80195520017.

## **BRAMSYS**

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**Sitio web:** www.bramsys.com.br

# USE INSTRUCTIONS

## STEREOTACTIC CANNULAS GUIDED SURGERY 14G-243

### PURPOSE

Stereotactic Cannulas are destined to stereotactic surgery procedures.

### INDICATIONS OF USE

- The Guided Surgery Cannula is suitable to serve as guidance in cerebral surgeries.
- The usage instructions that are within the product must be read, before opening the sterile package, assuring by that a correct and safe use.

### COMPOSITION

1 Cannula, 1 Stopper and 3 Adaptors. All parts and pieces that compose the product are commercialized to exclusive use with it.

### USE INSTRUCTIONS

- Be sure that the Stereotactic frame is correctly assembled on the patient;
- Be sure that the surgical planning of the stereotactic coordinates is ready and reviewed;
- Verify if the packing is not damaged or wet;
- Verify if the sterilization indication is correct;
- Verify the sterilization period of validity;
- Open the packing and remove the sealed blister;
- Following the procedures of handling sterile products, open the blister and remove the product;
- Find in the equipment the assembling place of the adaptors;
- Assemble the adaptors in the determined places, observing the correct order;
- Accomplish the pre assembly and adjust of the coordinates in the Stereotactic equipment defining and confirming that the cannula's end is in the target;
- The guided surgery cannula's length is adjusted in 243 mm, if you use in equipments that require different lengths, locate the stopper to attend the stereotactic coordinates in order to reach the desired target;
- Accomplish the definitive assembly of the Stereotactic on the patient and introduce the cannula delicately, through the adaptor's orifice, until reaching the target point in the brain;
- Fasten the fetter screw of the chariot to prevent the movement of the cannula;
- Accomplish the stereotactic navigation in accordance with the procedure that has to be done;
- After the procedure, unlock the cannula, remove it from the equipment, make it useless and discard it in HOSPITAL WASTE.

### PRECAUTIONS, RESTRICTIONS AND WARNINGS

- This product should only be used by a specialized professional and specifically trained in the accomplishment of the "Stereotactic Guided Surgery" procedure.
- The user, before using the product, should read carefully the use instructions that will aid him in the correct and safe use and will also protect against possible risks.
- Deformations in the stereotactic system structure may compromise the precision of the procedure, and it may cause harm to the patient's brain.
- Fasten the movables parts of the Stereotactic equipment adequately.
- Excessive effort to fasten the stereotactic frame in the skull, above the supported by the bony structure, may compromise the cranial case and occasion harm to the patient.
- The incorrect placement of the adaptors in the chariot may cause difference in the planned stereotactic coordinate, compromising the precision of the system what may cause harm to the patient brain.
- Do not use the product out of the validity date, always verify if the sterilization indication is correct.
- The product should only be used by qualified medical professionals, who have the necessary knowledge and experience for the use of the product, knowledge of the handling of sterile products and, mainly, of the importance of maintaining an atmosphere totally sterile in the surgery rooms, where the care and precaution with each detail are fundamental in order to avoid contamination risks.
- The product should not be reused and/or re-sterilized, being

appropriate for **ONLY ONE USE**. It must be destroyed and discarded in **HOSPITAL WASTE**.

- The product should be transported and always conditioned (kept) protected against every type of humidity, dirt and the contact with inflammable, corrosive, toxic or explosive substances.
- The packing must not be pierced or torn, it must be distant from cutting or sharp instruments.
- Do not keep or store this product close or near to products of different categories.
- Verify the integrity of the packing before using the product. In case of violation, **DO NOT USE**, because the warranty of the product sterility will be committed.
- It is prohibited to reprocess this product.
- This product is appropriate for only one use, sterilized by Ethylene Oxide (EO) or Gamma Radiation.

This product is very safe if used by qualified professionals and respecting the usage specifications described in this document.

**Lot number, Sterilization and Expiration Date:** refer to the product packing

**MS Registration:** 80195520017

**Technical Responsible:** Marcos Rojas

**CREA:** 5061214879

### **WARRANTY CERTIFICATE**

**BRAMSYS INDÚSTRIA E COMÉRCIO LTDA**, properly authorized by the National Agency of Sanitary Surveillance, guarantees for **24 MONTHS**, starting from the sterilization date, the quality and the sterile condition of this product, except in the cases of bad usage or noncompliance of the orientations written in this document.

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