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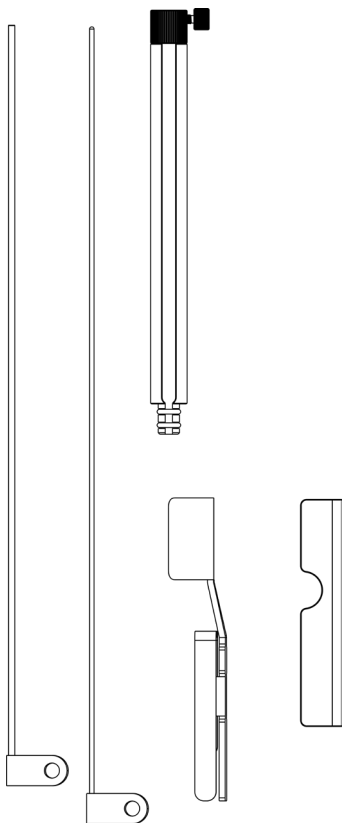
STEREOTACTIC CANNULAS DEEP ELECTRODE IMPLANTATION

PURPOSE

Stereotactic Cannulas are destined to stereotactic surgery procedures.

INDICATIONS OF USE

- The Deep Electrode Implantation Cannula is suitable for implanting electrodes in the cerebral region by stereotactic surgery.
- The usage instructions that are within the product must be read before opening the sterile package, ensuring a correct and safe use.

COMPOSITION

1 Cannula, 1 Mandrel, 1 Guide Frame, 2 Brackets and 3 Adaptors. All parts and pieces that compose the product are commercialized to exclusive use with it.

USE INSTRUCTIONS

- Be sure that the Stereotactic frame is correctly assembled on the patient;
- Be sure that the surgical planning of the stereotactic coordinates is ready and reviewed;
- Check that the packing is not damaged or wet;
- Check that the sterilization indication is correct;
- Check the expiration date of the sterilization;
- Open the packing and remove the sealed blister;
- Following the handling procedures of sterile products, open the blister and remove the product;
- Locate the place of assembly of the adaptors in the equipment;
- Assemble the adaptors in the determined places, observing the correct order;
- Complete the pre assembly and adjustments of the coordinates in the Stereotactic equipment defining and confirming that the cannula's end is in the target;
- Complete the definitive assembly of the Stereotactic on the patient and incise the guide frame in the Stereotactic delicately through the adaptor's orifice.
- Introduce the mandrel into the cannula, and the cannula into the guide frame until the cannula's brim leans against the adaptor;
- Remove the mandrel;
- Introduce the electrode through the guide frame and through the cannula, determining the length until the target;
- Lock the electrode in the target with the lower adapter and remove the cannula, careful not to move the electrode from its initial position;
- Lock the electrode with the stop from the guide frame, careful not to damage the electrode;
- Fit the acrylic bracket from the respective electrode in the guide frame;
- Fit the neurostimulator connector on the acrylic bracket;
- Complete the electrode stimulator test;
- After the procedure, discard the product in HOSPITAL WASTE.

PRECAUTIONS, RESTRICTIONS AND WARNINGS

- This product should only be used by a professional specialized and specifically trained in the "Stereotactic Deep Electrode Implantation Surgery" procedure.
- Before using the product, the user should read the use instructions carefully, which will aid him in the correct and safe use and will also protect against possible risks.
- The introduction of the cannula into the patient's brain should always be performed carefully and with the mandrel. Otherwise, it can cause serious damages to the patient's brain, including the risk of hemorrhage.
- Each type of material collected has a different consistence, so the quantity and the difficulty to collect it may change.
- Deformations in the structure of the stereotactic system may compromise the precision of the procedure, and may cause harm to the patient's brain.
- Fasten the movables parts of the Stereotactic equipment adequately.
- Excessive effort to fasten the stereotactic frame in the skull, above the supported by the bony structure, may compromise the cranial case and cause harm to the patient.
- The incorrect placement of the adaptors in the cart may cause

difference in the planned stereotactic coordinate, compromising the precision of the system and causing harm to the patient brain.

- Do not use the product out of the expiration date. Always check that the sterilization indication is correct.
- The product should only be used by qualified medical professionals, with the necessary knowledge and experience to use the product, knowledge of the handling of sterile products and, mainly, of the importance of maintaining a totally sterile atmosphere in the surgery room, where the care and precaution with each detail are fundamental in order to avoid contamination risks.
- The product should not be reused and/or resterilized, being appropriate for ONLY ONE USE. It must be destroyed and discarded in HOSPITAL WASTE.
- The product should be transported and always conditioned (kept) protected against every type of humidity, dirt and the contact with inflammable, corrosive, toxic or explosive substances.
- The packing must not be pierced or torn, it must be kept away from cutting or sharp instruments.
- Do not keep or store this product close or near to products of different categories.
- Check the integrity of the packing before using the product. In case of violation, DO NOT USE, because the warranty of the product's sterility will be committed.
- It is prohibited to reprocess this product.
- This product is appropriate for single use only, sterilized by Ethylene Oxide (EO) or Hydrogen Peroxide or Gamma Radiation.

This product is very safe if used by qualified professionals and respecting the usage specifications described in this document.

Lot number, Sterilization and Expiration Date: refer to the product packing

MS Registration: 80195520018

Technical Responsible: Marcos Rojas

CREA: 5061214879

WARRANTY CERTIFICATE

BRAMSYS INDÚSTRIA E COMÉRCIO LTDA, properly authorized by the National Agency of Sanitary Surveillance, guarantees for **24 MONTHS**, starting from the sterilization date, the quality and the sterile condition of this product, except in the cases of bad usage or noncompliance of the orientations written in this document.

CUSTOMER SERVICE (CS)

Phone: +55 11 5013 0010

E-mail: sac@bramsys.com.br

Site: www.bramsys.com.br