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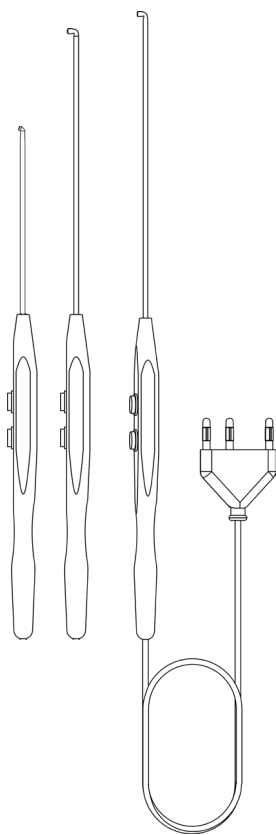
Reg. ANVISA: 80195520002.

BRAMSYS

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BRAMSYS ABLATION ELECTRODES COC, RIB AND PEQ

PURPOSE

Bramsys Ablation Electrodes are destined to produce controlled ablation, coagulation and tissue modification through radiofrequency in surgical procedures.

INDICATIONS OF USE:

- The Bramsys Ablation Electrodes are suitable for procedures of controlled ablation, coagulation and tissue modification during procedures of Arthroscopic surgery of ATM, arthroplasty of ATM and ATM arthroplasty with discopexy.
- The usage instructions that are within the product must be read before opening the sterile package, assuring by that a correct and safe use.

| Code | Series | Format | Useful Length | Angle | Manual Activation |
|----------|--------|----------|---------------|-------|-------------------|
| BMS-D022 | COC | Concave | 170mm | 90° | No |
| BMS-D005 | COC | Concave | 170mm | 90° | Yes |
| BMS-D011 | RIB | Striatum | 170mm | 90° | No |
| BMS-D006 | RIB | Striatum | 170mm | 90° | Yes |
| BMS-D118 | PEQ | Convex | 90mm | 60° | Yes |
| BMS-D120 | PEQ | Convex | 170mm | 60° | Yes |

USE INSTRUCTIONS

- Verify if the packing is not damaged or wet;
- Verify if the sterilization indication is correct;
- Verify the sterilization period of validity;
- Open the packing and remove the sealed blister;
- Following the procedures of handling sterile products, open the blister and remove the product;
- Connect the cable of the electrode to the electrosurgical unit which must be connected to the corresponding electrical net;
- Turn the electrosurgical unit on;
- COC and RIB: Adjust the ablation power (ablate) to 60W and the coagulation power (coag) to 30W;
- Perform the applicable surgical procedure;
- PEQ: Adjust the ablation (ablate) and coagulation (coag) power to 30W and 15W respectively;
- Accomplish the applicable surgical procedure;
- COC and RIB: If necessary, readjust the ablation (ablate) or coagulation (coag) powers to better adapt them to the type of procedure being performed, knowing that the maximum powers that should be used are 90W for ablation and 45W for coagulation;
- PEQ: If necessary, re-adjust the ablation (ablate) and coagulation (coag) power in accordance to the procedure in execution, knowing that the maximum power to be used are 50W for ablation e 25W for coagulation;
- After the procedure, turn the electrosurgical unit off, disconnect the electrode, make it useless and discard it in HOSPITAL WASTE.

PRECAUTIONS, RESTRICTIONS AND WARNINGS

- Bramsys ablation electrodes was developed to perfect compatibility with the unit Radiofrequency Generator System BMS-200M.
- The use of Bramsys electrodes with others Generator units may compromise their efficiency or durability. Always consult Bramsys customer service to specific orientation
- The user, before using the product, should carefully read the instructions that will aid him in the correct and safe use and will also protect against possible risks.
- When using rigid return plates (iron plates), always use conductive gel between the plate's surface of contact and the patient, putting it in order to get the greater possible area of contact between the plate and the patient.
- Do not handle or position the electrode over the patient or user while the radiofrequency is activated.
- Do not use in inflammable or explosive atmosphere.
- Do not use the product out of the validity date and always verify if the sterilization indication is correct.
- The product should only be used by qualified medical professionals, who have the necessary knowledge and experience for the use of the product, knowledge of the handling of sterile products and, mainly, of the importance of maintaining an atmosphere totally sterile in the surgery rooms, where the care and precaution with each detail are fundamental in order to avoid contamination risks.
- The product should not be reused and/or re-sterilized, being appropriate for ONLY ONE USE. It must be destroyed and discarded in HOSPITAL WASTE.
- The product should be transported and always conditioned (kept) protected against every type of humidity, dirt and contact with inflammable, corrosive, toxic or explosive substances.
- The packing must not be pierced or torn, it must be distant from cutting or sharp instruments.
- Do not to keep or store this product close or near to products of different categories.
- Maintain the product distant from cutting or sharp objects.
- Verify the integrity of the packing before using the product. In case of violation, DO NOT USE, because the warranty of its sterility will be committed.
- It is prohibited to reprocess this product.
- This product is appropriate for only one use, sterilized by Ethylene Oxide (EO) or Hydrogen Peroxide or Gamma Radiation.

This product is totally safe if used by qualified professional and respecting the usage specifications described in this document.

Lot number, Sterilization and Validity Date: see the product packing

MS Registration: 80195520002

WARRANTY CERTIFICATE

BRAMSYS INDÚSTRIA E COMÉRCIO LTDA, duly authorized by the National Health Surveillance Agency, guarantees for 24 MONTHS, from the sterilization date, the quality and sterile condition of this product, except in cases of misuse or non-compliance with the guidelines contained in this document.

SERVIÇO DE ATENDIMENTO AO CLIENTE (SAC)
SERVICIO DE ATENDIMIENTO AL CLIENTE (SAC)
CUSTOMER SERVICE (CS)
Telefone/Phone: +55 11 5013 0010
E-mail: sac@bramsys.com.br
Site: www.bramsys.com.br