
Kit Cãnula Guia

Reg. ANVISA: 80195520023.

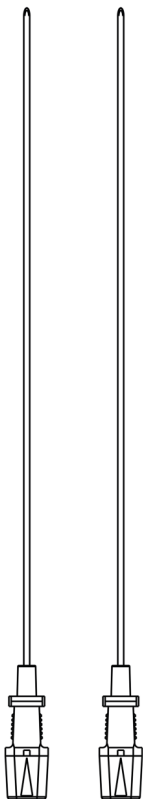BRAMSYS

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USE INSTRUCTIONS

Guide Cannula Kit

The Guide Cannulas are intended for the performance of surgical procedures in the treatment of pain.

INDICATIONS FOR USE

The Guide Cannulas are intended for the performance of surgical procedures in the treatment of pain, guiding the location of the target in minimally invasive surgical procedures.

The user must read the instructions for use that accompany the product before opening the sterile packaging, thus ensuring its correct and safe use.

COMPOSITION

Primary packaging: A surgical-grade paper envelope containing 1 to 8 cannulas of the same commercial model, wrapped in a plastic protective tube.

Secondary packaging: A cardboard box.

INSTRUCTIONS FOR USE

- Check that the packaging is not damaged or wet.
- Check that the sterilization indication is correct.
- Check the expiration date of the sterilization.
- Following the procedures for handling sterile products, open the packaging and remove the product.
- Remove the protective tube from the cannula.
- Check the correct positioning of the mandrel in the cannula.
- Insert the cannula into the desired location, locating the target.
- Remove the mandrel and perform the indicated surgical procedure.
- The product should not be reused or re-sterilized, and is for SINGLE USE only.
- After the procedure, dispose of the product in HOSPITAL WASTE.

PRECAUTIONS, RESTRICTIONS, STORAGE AND WARNINGS

- The user, before using the product, must carefully read the instructions for use, which will assist in the correct and safe use and protect against potential risks.
- Do not use the product after the expiration date, always check that the sterilization indication is correct.
- The product should only be used by qualified medical professionals who have the knowledge and experience necessary for the proper use of the product, knowledge of handling sterile products, and, most importantly, the importance of maintaining a completely sterile environment in operating rooms, where care and caution with every detail are essential to prevent contamination.
- The product should not be reused or re-sterilized, and is for SINGLE USE only, and must be destroyed and disposed of in HOSPITAL WASTE.
- The product must be transported and stored protected from all types of moisture, dirt and contact with flammable, corrosive, toxic or explosive substances.
- The packaging must not be pierced or torn, and must be kept away from sharp or pointed instruments.
- Do not store or store this product near products of different categories.
- Check the integrity of the packaging before using the product, in case of violation, DO NOT USE, as the guarantee of its sterility will be compromised.
- Reprocessing of this product is prohibited.
- This product is for single use, sterilized by Ethylene Oxide (ETO).
- Storage temperature: 5°C to 60°C.
- Storage humidity: 10% to 80%.

Product is completely safe if used by a qualified professional and respecting the use specifications described in this document.

Lot Number, Sterilization Date and Expiration Date: See product packaging.

MS Registration: 80195520023

Technical Responsible: Marcos Rojas

CREA: 5061214879

Code	Ø	Lenght	Active Tip	Quantity
CSC-2115010 (1)	21G	150mm	10mm curved	1
SSC-1805005 (1)	18G	50mm	5mm straight	1
SSC-1810005 (1)	18G	100mm	5mm straight	1
SSC-1815005 (1)	18G	150mm	5mm straight	1
SSC-1810010 (1)	18G	100mm	10mm straight	1
SSC-1815010 (1)	18G	150mm	10mm straight	1
SSC-2105005 (1)	21G	50mm	5mm straight	1
SSC-2110005 (1)	21G	100mm	5mm straight	1
SSC-2110010 (1)	21G	100mm	10mm straight	1
SSC-2110015 (1)	21G	100mm	15mm straight	1
SSC-2115005 (1)	21G	150mm	5mm straight	1
SSC-2115010 (1)	21G	150mm	10mm straight	1
SSC-2115015 (1)	21G	150mm	15mm straight	1
SSC-2205005 (1)	22G	50mm	5mm straight	1
SSC-2210005 (1)	22G	100mm	5mm straight	1
SSC-2210010 (1)	22G	100mm	10mm straight	1
SSC-2215005 (1)	22G	150mm	5mm straight	1
SSC-2215010 (1)	22G	150mm	10mm straight	1
CSC-2115010	21G	150mm	10mm curved	2
SSC-1805005	18G	50mm	5mm straight	2
SSC-1810005	18G	100mm	5mm straight	2
SSC-1815005	18G	150mm	5mm straight	2
SSC-1810010	18G	100mm	10mm straight	2
SSC-1815010	18G	150mm	10mm straight	2
SSC-2105005	21G	50mm	5mm straight	2
SSC-2110005	21G	100mm	5mm straight	2
SSC-2110010	21G	100mm	10mm straight	2
SSC-2110015	21G	100mm	15mm straight	2
SSC-2115005	21G	150mm	5mm straight	2
SSC-2115010	21G	150mm	10mm straight	2
SSC-2115015	21G	150mm	15mm straight	2
SSC-2205005	22G	50mm	5mm straight	2
SSC-2210005	22G	100mm	5mm straight	2
SSC-2210010	22G	100mm	10mm straight	2
SSC-2215005	22G	150mm	5mm straight	2
SSC-2215010	22G	150mm	10mm straight	2

WARRANTY CERTIFICATE

BRAMSYS INDÚSTRIA E COMÉRCIO LTDA, properly authorized by the National Agency of Sanitary Surveillance, guarantees for **24 MONTHS**, starting from the sterilization date, the quality and the sterile condition of this product, except in the cases of bad usage or noncompliance of the orientations written in this document.

CUSTOMER SERVICE (CS)

Phone: +55 11 5013 0010

E-mail: sac@bramsys.com.br

Site: www.bramsys.com.br