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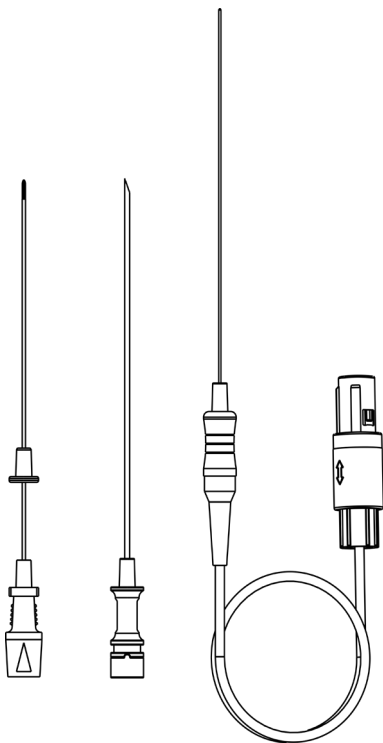
# USE INSTRUCTIONS

## INTRADISCAL ABLATION CANNULA KIT

Bramslys Ablation Cannulas are designed to guide electrodes that produce radiofrequency ablation in various types of surgical procedures.

## INDICATIONS FOR USE

- The Intradiscal Ablation Cannulas 18G-100 are specifically indicated for intradiscal ablation procedures in gelatinous tissue, particularly in the cervical region.
- The instructions for use that accompany the product must be read before opening the sterile packaging to ensure correct and safe use.

## COMPOSITION

1 Cannula, 1 Contrast Cannula, 1 Stopper, and 1 Electrode. All parts and components of the product are sold exclusively for use with it.

Code	Diâmetro	Comprimento
BMS-D569	18	100
BMS-D570	18	200

## INSTRUCTIONS FOR USE

- Check that the packaging is not damaged or wet,
- Verify that the sterilization indicator is correct,
- Check the sterilization expiration date,
- Open the packaging and remove the sealed product,
- Following sterile product handling procedures, open the second package and remove the product,
- Connect the electrode connection cable to the radiofrequency unit, ensuring it is properly connected to the appropriate power source,
- Connect the return cable to the radiofrequency unit and the disposable return pad,
- Apply the return pad to the patient, positioning it on a flat, clean area (thigh, buttocks, or upper back),
- Turn on the radiofrequency unit,
- Adjust the SETUP parameters (time and stimulation) according to the procedure,
- Using the impedance function, verify that the indication corresponds to the expected value for the electrode location,
- Perform the appropriate stimulation process,
- Carry out the ablation procedure, carefully adjusting the power to achieve the desired temperature,
- If necessary, repeat the steps above according to the protocol established for this procedure,
- After the procedure, turn off the radiofrequency unit, disconnect the electrode and return pad, remove the cannula, render these products unusable, and dispose of them in MEDICAL WASTE.

## PRECAUTIONS, RESTRICTIONS, STORAGE, AND WARNINGS

- Before using the product, the user must carefully read the instructions for use to ensure correct and safe operation and prevent potential risks.
- When using rigid return pads (steel plates), always apply conductive gel between the plate's contact surface and the patient, ensuring the largest possible contact area.
- Do not handle or position the electrode on the patient or user while the radiofrequency is active.
- Do not use in flammable or explosive environments.
- Do not use the product past its expiration date; always check that the sterilization indicator is correct.
- The product must only be used by qualified medical professionals with the necessary knowledge and experience for proper use, sterile product handling, and, most importantly, an understanding of the importance of maintaining a completely sterile environment in operating rooms. Every precaution and detail is crucial to avoid contamination risks.
- The product must not be reused or re-sterilized. It is for SINGLE USE ONLY and must be disposed of in MEDICAL WASTE after use.
- The product must be transported and stored protected from all types of moisture, dirt, and contact with flammable, corrosive, toxic, or explosive substances.
- The packaging must not be punctured or torn and should be kept

away from sharp or pointed instruments.

- Do not store or keep this product near different category products.
- Check the integrity of the packaging before use. In case of violation, DO NOT USE, as the sterility guarantee will be compromised.
- Reprocessing of this product is prohibited.
- This product is for single use only and is sterilized by Ethylene Oxide (ETO), Hydrogen Peroxide, or Gamma Radiation.

**Batch No., Sterilization Date, and Expiration Date:** See product packaging.

**MS Registration No.:** 80195520009

**Responsible Technician:** Marcos Rojas

**CREA:** 5061214879

#### **WARRANTY CERTIFICATE**

**BRAMSYS INDÚSTRIA E COMÉRCIO LTDA**, properly authorized by the National Agency of Sanitary Surveillance, guarantees for **24 MONTHS**, starting from the sterilization date, the quality and the sterile condition of this product, except in the cases of bad usage or noncompliance of the orientations written in this document.

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