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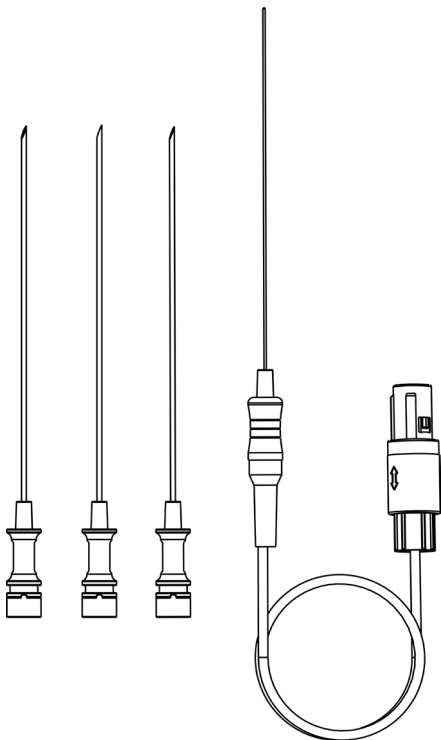
USE INSTRUCTIONS

CANNULA FOR TRIGEMINAL ABLATION

Bramslys Ablation Cannulas are designed to guide electrodes that produce radiofrequency ablation in various types of surgical procedures.

INDICATIONS FOR USE

- The Trigeminal Ablation Cannulas are specifically indicated for trigeminal nerve ablation procedures.
- Users must read the instructions for use that accompany the product before opening the sterile packaging to ensure correct and safe usage.

COMPOSITION

3 Cannulas and 1 Electrode. All parts and components of the product are marketed for exclusive use with it.

| Code | Diameter | Length |
|----------|----------|--------|
| BMS-D565 | 18G | 100mm |
| BMS-D651 | 21G | 100mm |

INSTRUCTIONS FOR USE

- Check that the packaging is not damaged or wet.
- Check that the sterilization indicator is correct.
- Check the sterilization expiration date.
- Open the package and remove the conditioned and sealed product.
- Following sterile product handling procedures, open the second package and remove the product.
- Connect the electrode's connection cable to the radiofrequency unit, which must be connected to the appropriate power supply.
- Connect the return cable to the radiofrequency unit and the disposable return plate.
- Apply the return plate to the patient, positioning it on a flat and clean area (thigh, buttocks, or upper back).
- Turn on the radiofrequency unit.
- Adjust the appropriate SETUP parameters (time and stimulation) for the procedure.
- Using the impedance function, verify that the indication matches the expected value for the electrode's location.
- Perform the appropriate stimulation process.
- Carry out the ablation procedure, carefully and properly adjusting the power to achieve the desired temperature.
- If necessary, repeat the above steps according to the protocol adopted for this procedure.
- After the procedure, turn off the radiofrequency unit, disconnect the electrode and the return plate, remove the cannula, disable these products, and dispose of them in MEDICAL WASTE.

PRECAUTIONS, RESTRICTIONS, AND WARNINGS

- Antes de utilizar el producto, el usuario debe leer atentamente las • Before using the product, the user must carefully read the instructions for use, which will assist in proper and safe usage and help prevent possible risks.
- When using rigid return plates (steel plates), always apply conductive gel between the plate's contact surface and the patient, positioning it to ensure the largest possible contact area.
- Do not handle or position the electrode on the patient or user while the radiofrequency is activated.
- Do not use in flammable or explosive atmospheres.
- Do not use the product beyond the expiration date; always check that the sterilization indicator is correct.
- The product should only be used by qualified medical professionals who have the necessary knowledge and experience for proper use, handling of sterile products, and, most importantly, the importance of maintaining a completely sterile environment in operating rooms, where care and attention to every detail are essential to prevent contamination risks.
- The product must not be reused or re-sterilized; it is for SINGLE USE ONLY and must be destroyed and disposed of in MEDICAL WASTE.
- The product must always be transported and stored protected from

any type of moisture, dirt, and contact with flammable, corrosive, toxic, or explosive substances.

- The packaging must not be punctured or torn and should be kept away from sharp or pointed instruments.
- Do not store this product near products of different categories.
- Check the integrity of the packaging before use; if it is compromised, DO NOT USE, as the sterility guarantee will be affected.
- Reprocessing this product is strictly prohibited.
- This product is for single use only and is sterilized using Ethylene Oxide (ETO), Hydrogen Peroxide, or Gamma Radiation.

Batch No., Sterilization Date, and Expiration Date: see product packaging

MS Registration No.: 80195520009

Technical Responsible: Marcos Rojas

CREA: 5061214879

WARRANTY CERTIFICATE

BRAMSYS INDÚSTRIA E COMÉRCIO LTDA, properly authorized by the National Agency of Sanitary Surveillance, guarantees for **24 MONTHS**, starting from the sterilization date, the quality and the sterile condition of this product, except in the cases of bad usage or noncompliance of the orientations written in this document.

CUSTOMER SERVICE (CS)

Phone: +55 11 5013 0010

E-mail: sac@bramsys.com.br

Site: www.bramsys.com.br