
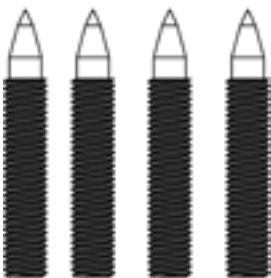
Parafusos Estereotáxicos

Reg. ANVISA: 80195520006.

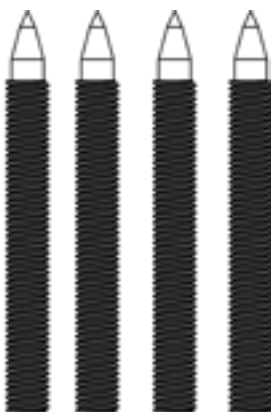BRAMSYS

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M8x75 SCREW

PURPOSE

The Bramsys Screws are intended to fasten the skull to surgical equipment.

INDICATIONS FOR USE

- The Bramsys Screws are suitable for surgical procedures where fastening the skull to surgical, particularly stereotactic, equipment is required;
- The instructions for use that come with the product must be read before opening the sterile package, thereby ensuring an appropriate and safe use.

COMPOSITION

Screws. All the parts that make up the product are sold exclusively for use therewith.

INSTRUCTIONS FOR USE

- Check that the package is not damaged or wet;
- Check that the sterilization signage is correct;
- Check the sterilization expiration date;
- Open the package and remove the product;
- According to the handling procedures for sterile products, open the blister and remove the screws;
- Adjust the screws in the stereotactic frame's holes;
- Place the stereotactic frame on the patient skull while observing the correct centralization and alignment with the skull;
- By using a 4 mm Hex key, fasten the screws to the patient's skull while observing the appropriate torque;
- Make sure that the screws don't go over the frame's parallels;
- Carry out the next steps of the surgical procedure;
- After the procedure, remove the screws from the equipment, render them useless and dispose of them in MEDICAL WASTE containers.

PRECAUTIONS, RESTRICTIONS, STORAGE, AND WARNINGS

- Overfastening the stereotactic frame in the skull, in excess of the bone structure strength, may compromise the braincase and harm the patient;
- The screws are compatible with the Bramsys stereotactic frame;
- For compatibility with other frames, please refer to Bramsys Customer Service;
- Before using the product, the user must carefully read the instructions, which will help him or her use it correctly and safely and protect him or her against potential risks;
- Do not use the product after the expiration date and always check that the sterilization signage is correct;
- The product must only be used by qualified medical professionals with the necessary knowledge and experience in using this product and conversant with the handling of sterile products and, mainly, the importance of keeping a totally sterile environment in operating rooms, where care and precaution with each detail are paramount, in order to avoid contamination risks;
- The manufacturer recommends that this product be used only once;
- This product must be transported and protected against any and all types of humidity, dirt, and contact with flammable, corrosive, toxic, or explosive substances;
- The package must not be pierced or torn. It must be kept away from cutting or sharp instruments;

Do not to keep or store this product near products of different categories;

Check the integrity of the package before using the product. In the event of violation, DO NOT USE IT because its sterility warranty will be compromised;

This product must be used only once. It is sterilized with ethylene oxide (ETO) or hydrogen peroxide or gamma radiation.

This product is absolutely safe if used by qualified professionals

in accordance with the specifications for use described herein.

Lot #, Sterilization and Expiration Date: see product package.

MS Registration: 80195520006

TIC: Marcos Rojas

CREA: 5061214879

WARRANTY CERTIFICATE

BRAMSYS INDÚSTRIA E COMÉRCIO LTDA, properly authorized by the National Agency of Sanitary Surveillance, guarantees for **24 MONTHS**, starting from the sterilization date, the quality and the sterile condition of this product, except in the cases of bad usage or noncompliance of the orientations written in this document.

CUSTOMER SERVICE (CS)

Phone: +55 11 5013 0010

E-mail: sac@bramsys.com.br

Site: www.bramsys.com.br